PermaCem

Dual

Inhalt | Contents

Gebrauchsinformation – DEUTSCH	4
Instructions for use – ENGLISH	11
Mode d'emploi – FRANÇAIS	18
Istruzioni per l'uso – ITALIANO	26
Instrucciones de uso – ESPAÑOL	34
Instruções de utilização – PORTUGUÊS	42
Gebruiks informatie – NEDERLANDS	50
Brugsanvisning – DANSK	58
Bruksanvisning – SVENSKA	65
Instrukcja użycia – POLSKI	72
Информация по применению – РУССКИЙ	80

Produktbeschreibung

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Product Description

PermaCem-Dual is a universal, adhesive, dual-curing compomer dental cement for cementing indirect restorations in combination with an enamel-dentine bonding agent such as LuxaBond (total-etch system) or Contax + Activator (self-etch system).

Indications

Permanent, adhesive cementation of crowns, bridges, inlays, onlays and veneers composed of

- Metal,
- Ceramic,
- Composite.

Contra-indications

- The use of PermaCem-Dual with dentine single-bottle bonding agents is contra-indicated.

- Do not use PermaCem-Dual in cases of known allergies to the substances contained or, as can occur in rare cases, in the event of contact allergy.
- The use of PermaCem-Dual with oxide ceramics is contra-indicated if no parallel-wall preparation is possible.

Basic Safety Instructions

- ▶ For dental use only!
- ▶ Keep out of reach of children!
- ▶ Avoid contact with the skin! In the event of inadvertent skin contact, wash the affected area thoroughly with soap and water.
- ▶ Avoid contact with eyes! In the event of inadvertent contact with eyes, immediately rinse the eyes thoroughly with plenty of water and consult a physician if necessary.
- ▶ In the event of application to dentine in close proximity to pulp, use appropriate pulp protection.

- ▶ In the event of allergic reactions, remove the material immediately and consult a physician, if necessary.

Side effects

There have been no systemic side effects to date. Isolated cases of hypersensitivity to components of the material cannot be excluded.

Interactions with other materials

Do not use PermaCem-Dual together with materials containing eugenol. DMG recommends eugenol-free temporary cements (e.g. TempoCemNE).

Application systems

- ▶ PermaCem Automix Dual: see “Using the Automix cartridge” on page 14.
- ▶ PermaCem Smartmix Dual: see “Using the Smartmix syringe” on page 15.

Timing

Processing time (including mixing time)*	≈ 2 min
Intraoral curing	5 min
Light curing	20 sec

*The time stated applies at a room temperature of 23 °C. Higher temperatures may shorten these times, while lower temperatures may increase them.

Instructions for use

- ▶ Cementation before combined work such as telescope work: Using a suitable separating agent, ensure that PermaCem-Dual cannot come between primary and secondary parts.
- ▶ Light-curing units should have an output of 450 nm and should be checked regularly. The light intensity should be a minimum of 400 mW/cm². Place the light as close as possible to the material. Observe the manufacturer’s instructions for the light unit.

Recommended Use

1. Prepare the clean and dry restoration as follows:
 - ▶ Etchable silicate ceramics: Etch the restoration with 5% hydrofluoric acid in accordance with the instructions from the manufacturer of the etching gel and rinse thoroughly with water spray. Subsequently dry with oil- and water-free air, and prepare the surface using a silanizing agent (e.g. DMG Silane) in accordance with the manufacturer's instructions.
 - ▶ Metals and metal alloys: Pretreatment in accordance with the manufacturer's instructions. Unless otherwise stated, sandblast the inside of the restoration with aluminum oxide, clean with alcohol and dry with oil- and water-free air.
 - ▶ Oxide ceramics: Pretreatment in accordance with the manufacturer's instructions. Ensure that there is sufficient parallel-wall retention.
2. Thoroughly remove any residue of the temporary cement from the prepared tooth


using water spray and dry with oil- and water-free air. When doing so, ensure that the dentine retains a little residual moisture and does not become overly dry.

Note: The use of rubber dam is recommended.

3. Apply a suitable bonding agent to the prepared tooth according to manufacturer's instructions. DMG recommends LuxaBond (total etch system) or Contax + Activator (self-etch system).
4. Apply PermaCem-Dual in the pre-treated restoration or on the prepared tooth.
5. Insert the restoration within 2:00 min of the start of mixing and attach it to the prepared tooth using slight pressure.
6. Instruct the patient to adopt habitual occlusion.
7. Remove excess cement within 1:30 to 2:00 min of inserting the restoration with the aid of a microbrush, brush, foam pellet or scaler. Remove interdental excess by flossing carefully.

8. Cure material with a suitable light unit (e.g. LED or halogen lamp) for at least 20 seconds. Then the restoration is initially fixed. After 5:00 minutes, the restoration has fully regained its capacity to withstand stress.

Using the Automix cartridge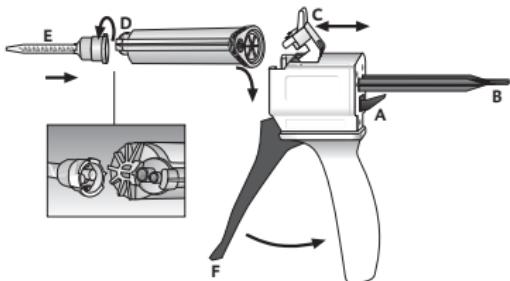

Insert cartridge

1. Push up the lever [A] at the back of the Automix-Dispenser and pull back the slide [B] completely.

2. Lift up the plastic lever [C], insert the cartridge and lock it using the plastic lever [C].

Note: Make sure that the notches on the cartridge line up with those on the Automix-Dispenser.

Attach mixing tip

1. Remove the cartridge cap [D] or used mixing tip by turning it 90° counterclockwise, and discard it.
2. Attach a new mixing tip [E].

Note: In order to attain an optimum mixing result, DMG recommends the use of the mixing tips available from DMG. All mixing tips are also available as refill packs.

Note: Make sure that the notches on the mixing tip line up with those on the cartridge.

3. Turn the mixing tip clockwise 90° to lock it.

Applying the material

- When using a cartridge for the first time, squeeze out a pea-sized amount of material and discard it.


- The material is mixed in the mixing tip by pulling the trigger [F] of the Automix-Dispenser and it is then applied directly.

Note: After the application, leave the used mixing tip on the cartridge to cap unused material!

Remove cartridge

- Push up the lever [A] at the back of the Auto-mix-Dispenser and pull back the slide [B] completely.
- Lift up the plastic lever [C] and remove the cartridge.


- Before attaching the mixing tip, remove the cap [A] or used mixing tip after turning it 90° counterclockwise, and discard it.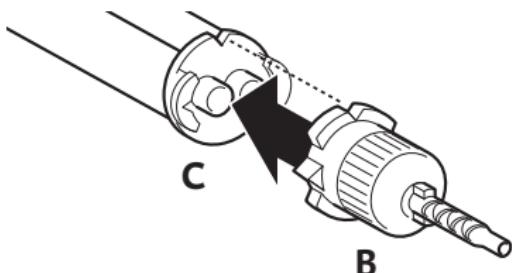

- Attach a new mixing tip [B].


Note: In order to attain an optimum mixing result, DMG recommends the use of the mixing tips available from DMG. All mixing tips are also available as refill packs.

Using the Smartmix syringe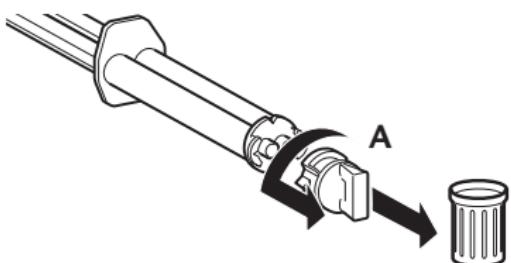

Note: Make sure that the notches on the mixing tip line up with those on the Smartmix injector [C].

- Turn the mixing tip clockwise 90° to lock it.

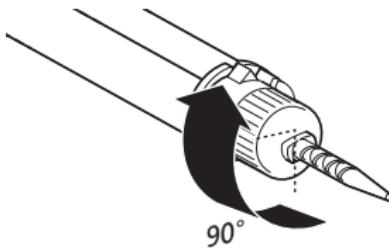4. The material is mixed in the cannula when it is expressed and it can be applied directly.

Note: When using an injector for the first time, squeeze out a pea-sized amount of material and discard it!

Note: Leave the used mixing cannula on the Smartmix syringe after use. It serves as a cap.

Disinfection of the Automix-Dispenser

- The Automix-Dispenser is reusable after disinfection, although it should be replaced if visible signs of wear appear. Disinfection can be carried out using a commercially available

dipping disinfectant. The Automix-Dispenser is also autoclavable.

- Dirty or contaminated mixing cannulas used as a cap can be cleaned with disinfectants normally used in practice.
- DMG recommends using only disinfecting solutions listed by the RKI (Robert Koch Institute).

Storage and shelf life

- ▶ Store at 2–25 °C/36–77 °F in a dry place.
- ▶ Do not use after the expiration date.

Composition

Ionomer glass in a Bis-GMA-based matrix consisting of dental resins, activators and additives.

Packaging

PermaCem-Smartmix Dual

2 syringes containing 10 g paste 20 Smartmix-Tips	REF 212035*
---	-------------

*In the USA and Canada, 10 Smartmix-Tips are included.

PermaCem-Automix Dual

1 cartridge containing 52 g paste 35 Automix-Tips	REF 110524**
1 cartridge containing 52 g paste 40 Automix-Tips	REF 110525***

** Not available in the USA and Canada

***Only available in the United States and Canada

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DMG Chemisch-Pharmazeutische Fabrik GmbH