


## Prepare teeth for bonding with Fiber-Bond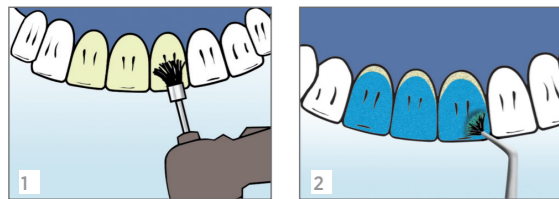


## Coat teeth with Fiber-Bond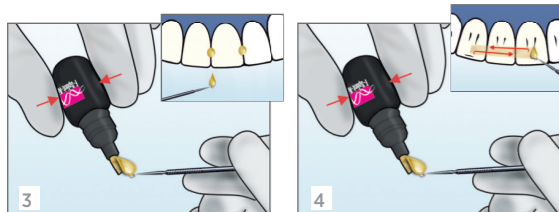

## Prepare the fiber-glass strip

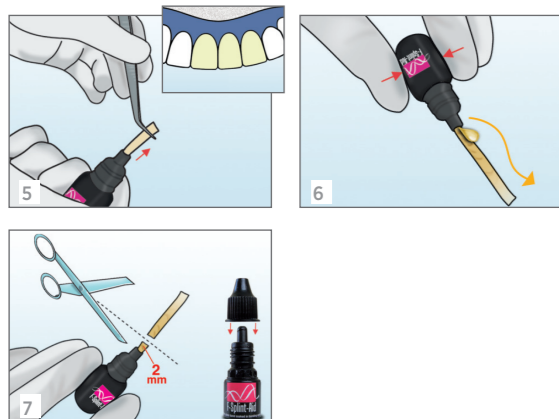## Place the fiber-glass strip on the teeth, light-cure and finish
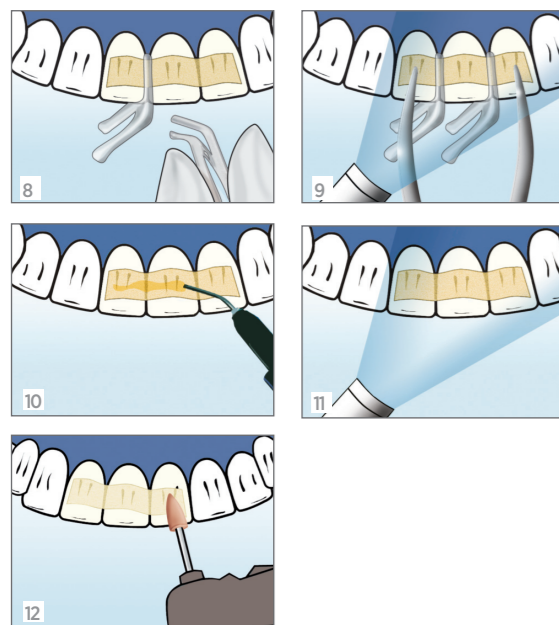
## English - F-Splint-Aid & Slim

Fiberglass strip soaked in Fiber-Bond agent. Suitable for 4-7 applications. Indicated for splinting of traumatically or periodontally loose anterior teeth, re-implantation of totally luxated anterior teeth (original extracted tooth or built up with composite). F-Splint-Aid & Slim is also suitable to reinforce temporary bridges or for post-orthodontic retainers.

### INSTRUCTIONS FOR USE

#### Prepare teeth for bonding with Fiber-Bond

1. Clean all involved teeth with a non-fluoride paste and place a rubber dam.
2. Etch the lingual/palatal and interdental surfaces with Fiber-Etch (REF 5991).

#### Coat teeth with Fiber-Bond

3. Extremely loose teeth can be fixed temporarily, by applying a small amount of Fiber-Bond or Fiber-Flow (REF 5991) interdentially, from the buccal side.
4. Squeeze the F-Splint-Aid & Slim bottle to saturate an applicator with Fiber-Bond. Coat the lingual/palatal and interdental surfaces with it.

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#### Prepare the glass fiber strip

5. Pull the appropriate strip length from the F-Splint-Aid & Slim bottle with tweezers.
6. Squeeze the bottle to well saturate the glass fiber strip with Fiber-Bond.
7. Cut the soaked strip. Leave at least a 2 mm long strip for next use. Close F-Splint-Aid & Slim bottle tightly after use, in order to preserve the product.

#### Place the glass fiber strip on the teeth, light-cure and finish

8. Place strip onto the lingual/palatal surface and hold it in place with the application clips (Clip&Splint).
9. Keep in place the ends of the strip with tweezers while polymerizing each interproximal space (30-40 sec.).
- 10-12. Any rough areas can be smoothed with a layer of Fiber-Flow (REF 5991). The splint is then trimmed and polished. Remove the rubber dam and check the occlusion.

**Important:** Instruct your patient about proper cleaning technique and prophylaxis.

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## splinting

## Reorder now

REF 5972	F-Splint-Aid: Fiber-Splint ribbon (4 mm wide, 120 mm long) impregnated with Fiber-Bond + 5 Clip&Splint application clips
REF 5977	F-Splint-Aid Slim: Fiber-Splint ribbon (2 mm wide, 120 mm long) impregnated with Fiber-Bond + 5 Clip&Splint application clips
REF 5980	F-Splint-Aid & Slim: REF 5972 + REF 5977
REF 5975CS	Clip&Splint, autoclavable application clips, 30 pcs
REF 5991	Fiber-Etch, Bond & Flow: 1 Fiber-Etch (syringe 3g + 10 tips) + 1 Fiber-Flow (syringe 2g + 10 tips) + 1 Fiber-Bond (bottle 3ml)

Only for professional use  
REV. 03 04.2017

**Polydentia SA**  
via Cantonale 47 • 6805 Mezzovico-Vira • Switzerland  
Phone +41 91 946 29 48 • Fax +41 91 946 32 03  
[www.polydentia.ch](http://www.polydentia.ch) • [info@polydentia.ch](mailto:info@polydentia.ch)  
Made in Switzerland

Indicated for splinting of traumatically or periodontally loose anterior teeth, re-implantation of totally luxated anterior teeth (original extracted tooth or built up with composite). F-Splint-Aid & Slim is also suitable to reinforce temporary bridges or for post-orthodontic retainers.

- Non-invasive
- Aesthetic and comfortable for the patient
- Reliable, flexible and time-saving
- Outstanding results
- Unique bottle dispenser

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