PermaCem

Inhalt | Contents

Gebrauchsinformation – DEUTSCH	4
Instructions for use – ENGLISH	11
Mode d'emploi – FRANÇAIS	18
Istruzioni per l'uso – ITALIANO	26
Instrucciones de uso – ESPAÑOL	34
Instruções de utilização – PORTUGUÊS	41
Gebruiksaanwijzing – NEDERLANDS	48
Brugsanvisning – DANSK	55
Bruksanvisning – SVENSKA	62
Instrukcja użycia – POLSKI	69
Информация по применению – РУССКИЙ	77

Produktbeschreibung

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Product Description

PermaCem is a universal, adhesive, self-curing compomer dental cement for cementing indirect restorations in combination with an enamel-dentine bonding agent such as LuxaBond (total-etch system) or Contax + Activator (self-etch system).

Indications

Permanent, adhesive cementation of crowns, bridges, inlays, onlays and veneers composed of

- Metal,
- Ceramic,
- Composite.

Contra-indications

- The use of PermaCem with dentine single-bottle bonding agents is contra-indicated.
- Do not use PermaCem in cases of known allergies to any of the substances contained

or, as can occur in rare cases, in the event of contact allergy.

- The use of PermaCem with oxide ceramics is contra-indicated if no parallel-wall preparation is possible.

Basic Safety Instructions

- ▶ For dental use only!
- ▶ Keep out of reach of children!
- ▶ Avoid contact with the skin! In the event of inadvertent skin contact, wash the affected area thoroughly with soap and water.
- ▶ Avoid contact with eyes! In the event of inadvertent contact with eyes, immediately rinse the eyes thoroughly with plenty of water and consult a physician if necessary.
- ▶ Cover dentine close to the pulp with pulp protection. If applicable, apply pulp protection before creating an impression.
- ▶ In the event of allergic reactions, remove the material immediately and consult a physician, if necessary.

Side effects

There have been no systemic side effects to date. Isolated cases of hypersensitivity to components of the material cannot be excluded.

Interactions with other materials

Do not use PermaCem together with materials containing eugenol. DMG recommends eugenol-free temporary cements (e.g. TempoCemNE).

Application systems

- ▶ PermaCem Automix: see “Using the Automix cartridge” on page 14.
- ▶ PermaCem Smartmix: see “Using the Smartmix syringe” on page 15.

Timing

Working time (including mixing time)*	≈ 2 min
Intraoral curing	5 min

*The time stated applies at a room temperature of 23 °C. Higher temperatures may shorten these times, while lower temperatures may increase them..

Instructions for use

Cementation before combined work such as telescope work: Using a suitable separating agent, ensure that PermaCem cannot come between primary and secondary parts.

Recommended Use

1. Prepare the clean and dry restoration as follows:
 - ▶ Etchable silicate ceramic: Etch the restoration with 5% hydrofluoric acid in accordance with the instructions from the manufacturer and rinse thoroughly with water spray.


Subsequently dry with oil- and water-free air, and pre-treat the surface using a suitable silanizing agent (e.g. DMG Silane) in accordance with the manufacturer's instructions.

- ▶ Metals and metal alloys: Pretreatment in accordance with the manufacturer's instructions. Unless otherwise stated, sandblast the inside of the restoration with aluminum oxide, clean with alcohol and dry with oil- and water-free air.
 - ▶ Oxide ceramics: Pretreatment in accordance with the manufacturer's instructions. Ensure that there is sufficient parallel-wall retention.
2. Remove any residue of the temporary cement from the prepared tooth, clean thoroughly with water spray and dry with oil- and water-free air. When doing so, ensure that the dentin retains a little residual moisture and does not become overly dry.

Note: The use of rubber dam is recommended.

3. Apply a suitable bonding agent to the prepared tooth according to manufacturer's instructions. DMG recommends LuxaBond (total etch system) or Contax + Activator (self-etch system).
4. Apply PermaCem in the pre-treated restoration or on the prepared tooth .
5. Insert the restoration within 2:00 min of the start of mixing and attach it to the prepared tooth using slight pressure.
6. Instruct the patient to adopt habitual occlusion.
7. Remove excess cement within 1:30 to 2:00 min of inserting the restoration with the aid of a microbrush, brush, foam pellet or scaler. Remove interdental excess by flossing carefully.
8. Leave material to cure. After 5:00 minutes, the restoration has fully regained its capacity to withstand stress.

Using the Automix cartridge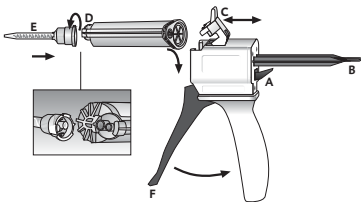

Insert cartridge

1. Push up the lever [A] at the back of the Automix-Dispenser and pull back the slide [B] completely.
2. Lift up the plastic lever [C], insert the cartridge and lock it using the plastic lever [C].

Note: Make sure that the notches on the cartridge line up with those on the Automix-Dispenser.

Attach mixing tip

1. Remove the cartridge cap [D] or used mixing tip by turning it 90° counterclockwise, and discard it.
2. Attach a new mixing tip [E].

Note: In order to attain an optimum mixing result, DMG recommends the use of the mixing tips available from DMG. All mixing tips are also available as refill packs.

Note: Make sure that the notches on the mixing tip line up with those on the cartridge.

3. Turn the mixing tip clockwise 90° to lock it.

Applying the material


- ▶ When using a cartridge for the first time, squeeze out a pea-sized amount of material and discard it.
- ▶ The material is mixed in the mixing tip by pulling the trigger [F] of the Automix-Dispenser and it is then applied directly.

Note: After the application, leave the used mixing tip on the cartridge to cap unused material!


Remove cartridge

1. Push up the lever [A] at the back of the Automix-Dispenser and pull back the slide [B] completely.
2. Lift up the plastic lever [C] and remove the cartridge.

Using the Smartmix syringe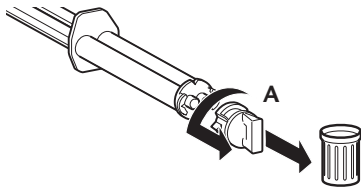

1. Before attaching the mixing tip, remove the cap [A] or used mixing tip after turning it 90° counterclockwise, and discard it.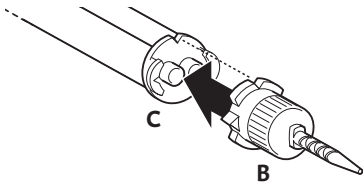


2. Attach a new mixing tip [B].

Note: In order to attain an optimum mixing result, DMG recommends the use of the mixing tips available from DMG. All mixing tips are also available as refill packs.

Note: Make sure that the notches on the mixing tip line up with those on the Smartmix injector [C].

3. Turn the mixing tip clockwise 90° to lock it.

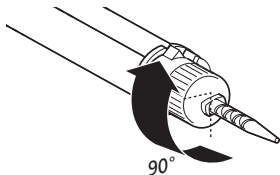4. The material is mixed in the cannula when it is expressed and it can be applied directly.

Note: When using an injector for the first time, squeeze out a pea-sized amount of material and discard it!

Note: Leave the used mixing cannula on the Smartmix syringe after use. It serves as a cap.

Disinfection

- The Automix-Dispenser is reusable after disinfection, although it should be replaced if visible signs of wear appear. Disinfection can be carried out using a commercially available dipping disinfectant. The Automix-Dispenser is also autoclavable.
- Dirty or contaminated mixing cannulas used as a cap can be cleaned with disinfectants normally used in practice.
- DMG recommends using only disinfecting solutions listed by the RKI (Robert Koch Institute).

Storage and shelf life

- ▶ Store at 2 – 25 °C/36 – 77 °F in a dry place.
- ▶ Do not use after the expiration date.

Composition

Ionomer glass in a Bis-GMA-based matrix consisting of dental resins, activators and additives.

Packaging

PermaCem-Smartmix	
2 syringes containing 10 g paste 20 Smartmix-Tips	REF 212034*

PermaCem-Automix	
1 cartridge containing 52 g paste 35 Automix-Tips	REF 110293*

* Not available in the USA and Canada

Accessories	
1 Automix-Dispenser type 25 1:1	REF 110253
50 Automix-Tips Blue 1:1	REF 110266
100 Automix-Tips Blue 1:1	REF 110267
50 Smartmix-Tips Short	REF 212040

Only available in the United States and Canada

Starter pack	
1 cartridge containing 52 g PermaCem-Automix, 1 Automix-Dispenser, 50 Automix-Tips	REF 110292

Refill pack	
1 cartridge containing 52 g PermaCem-Automix, 40 Automix-Tips	REF 110472

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DMG Chemisch-Pharmazeutische Fabrik GmbH