PREDCROWN

Mode d'emploi

MD UE Dispositif médical

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MD UE Medical device

Product description

PREDCROWN is a semi-permanent crown and bridge material based on multifunctional methacrylic esters. PREDCROWN is used for creating short- and long-term temporary crowns or bridges, inlays, onlays and veneers.

PREDCROWN is free of methyl methacrylate. Its temperature derived while curing is lower than 40 °C / 104 °F. As a temporary crown or bridge it protects the prepared teeth against external influences and preserves the occlusion. The cured temporary shows very high flexural strength, good abrasion resistance, low polymerization shrinkage and perfect fit. PREDCROWN can be polished to high gloss, is color stable and fluorescent in UV light.

And if PREDCROWN breaks - which may happen with acrylics - it is easy to repair. Cured material can be repaired with any natural or light cure composite building up again a chemical link. In nearly every property PREDCROWN is superior to the common materials based on monomer/polymer systems. It allows thin edges and production of longer bridge spans.

Indications/intended use

Fabrication of short- and long-term temporary crowns, bridges, inlays, onlays and veneers.

Performance features

The performance features of the product meet the requirements of the intended use.

Contraindications

Do not use for patients with allergic reactions against acrylates. PREDCROWN should not be used any more, if allergic reactions are observed.

Patient target group

Persons who are treated during a dental procedure.

Intended users

This medical device should only be used by a professionally trained dental practitioner.

Application

1. Impression taking

Before preparing of stumps or before a planned extraction of a tooth a situation impression is made by addition curing silicones (long storage stability!) or by alginates. For better stability of the temporary carve out interdental areas. In molar areas with teeth absent it may be necessary to cut a groove in the impression between the abutments to create a bridge-like connection between the tooth units.

Note:

Block out undercuts and if necessary cut grooves into the impression.

2. Preparing of the cartridge

The turn cap of the cartridge is taken off and thrown away (do not use it again!). Attach one of the supplied mixing cannulas and turn the cannula 90° clockwise until it locks in position. The application gun is loaded with the prepared cartridge and is ready for application.

Note:

The initial extrusion from the mixing cannula (about the size of a pea) should be discarded. Then the following mix will be perfect. This must be done for each new mix. Leave the used mixing cannula on the cartridge. It serves as a cap.

3. Application

PREDCROWN is automatically mixed when dispensed with slight and even pressure directly into the situation impression made before. Filling should occur from bottom upward to prevent voids.

4. Forming of the temporary crowns or bridges

- Load the situation impression with PREDCROWN.
- Seat the impression within the working time (40 seconds at 23 °C / 73 °F) onto the prepared teeth.

After 1-2 minutes (setting time in mouth at 37 °C / 99 °F) the material shows a hardened but still elastic condition and can be removed from the teeth together with the situation impression. (alternatively: 3-4 minutes setting time on the model)

Note:

The setting reaction has to be checked with excess material intra-orally (e.g. with a dental probe) resp. on the model. The oral temperature has a significant effect on the setting reaction and the temporary can only be removed without destruction during the elastic state.

5. Post curing and finishing

If possible, leave the temporary in the situation impression during post-curing.

Optimally the temporary is post-cured in warm water (45-55 °C / 113-131 °F, e.g. in a hot cure polymerization device) until reaching its final hardness (approx. 4:30 minutes from start of mixing). Post-curing at room temperature is completed after about 6 minutes from start of mixing.

When the cure is completed remove the temporary. Afterwards remove excess material and proximal undercuts. Then the temporary can be refined with rotary instruments and polished to high gloss.

Do not breathe polishing dust; use suitable mouth protective device, safety glasses and aspiration!

Note:

The oxygen inhibited smear layer on the surface should be removed before finishing. It can easily be removed by alcohol or other suitable solvents.

6. Cementing of the temporary

PREDCROWN temporary crowns or bridges preferably should be cemented with an eugenol-free temporary cement. In case of using eugenol-containing cements it should be considered that later used acrylate based cements could be hindered in hardening (inhibition of curing reaction by traces of remained eugenol).

7. Repairs of the temporary

PREDCROWN temporaries show high mechanical strength. However, if a temporary breaks the following procedures are recommended:

a. Fracture shortly after production

Both ends of the fracture are refixed with freshly extruded PREDCROWN.

b. Fracture of longer existing temporary

The areas of fracture are cleaned and roughened and provided with some mechanical retentions. Thus prepared, the fractured areas can be joint with freshly extruded PREDCROWN. To enhance the curing place for some minutes in warm water.

A flowable light cure composite can be applied alternatively.

8. Special hints

Non hardened PREDCROWN can easily be removed with alcohol or other suitable solvents.

9. Storage

Do not store above 25 °C / 77 °F!

Do not use after expiry date!

10. Additional notes/Warnings

- PREDCROWN is free of methyl methacrylate but contains other acrylates.
- Avoid contact with skin, mucous membrane and eyes.
- If the material comes into contact with skin, immediately wash with water and soap. If the material comes into contact with eyes, immediately rinse with copious amounts of water and seek medical advice if required.
- For sensitive persons, sensitization to the material cannot be excluded. The material should not be used any more, if allergic reactions are observed. Do not use for persons with allergic reactions against acrylates.
- Commercial medical gloves do not protect against the sensitizing effect of methacrylates.
- Keep away from children!

Composition

Unsaturated esters, multifunctional methacrylates, multifunctional acrylates, malonyl urea derivatives, dental glass, silicon dioxide, catalysts.

Disposal

Disposal of the product according to local authority regulations.

Reporting obligation

Serious incidents according to the EU Medical Devices Regulation that have occurred in connection with this medical device must be reported to the manufacturer and the competent authority.

Note

The summary of safety and clinical performance of the medical device can be found in the European database on medical devices (EUDAMED – <https://ec.europa.eu/tools/eudamed>).

Warranty


First Scientific Dental Materials GmbH warrants this product will be free from defects in material and manufacture. First Scientific Dental

Materials GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and First Scientific Dental Materials GmbH's sole obligation shall be repair or replacement of the First Scientific Dental Materials GmbH product.

Limitation of Liability

Except where prohibited by law, First Scientific Dental Materials GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

Time table of applicationn and curing of PREDCROWN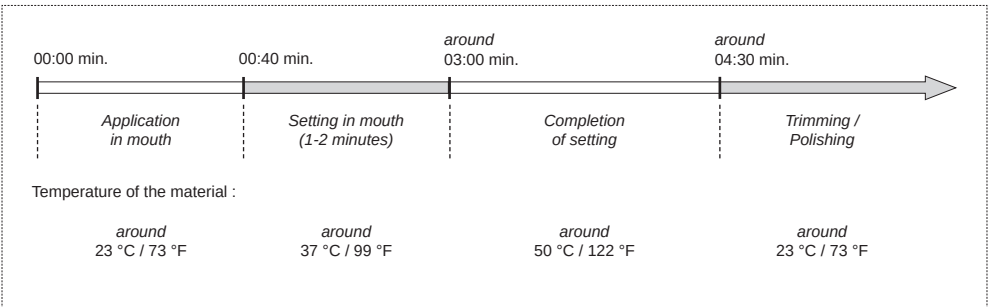

Note : If the completion of setting takes place at room temperature, trimming / polishing could be done not before after 6 minutes (from start of mixing).