Impregneurs PREDPOST+

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Instructions for use PREDPOST+ sleeves

Description of product

The sleeve is a device allowing the homogeneous impregnation of PREDPOST+. The sleeve is composed of FEP.

Indications

Using for endodontically treated teeth post and core build-up restorations.

Various sizes are available to meet all practitioners' needs.

Sheath color	Red	Blue	Black	Yellow
PREDPOST+	4	6	9	12

Contra-indications

In the current state of knowledge, there is no contra-indication.

Side effects

In the current state of knowledge, there is no contra-indication.

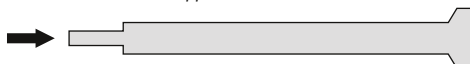
Precautions

- Avoid direct contact with fingers.
- The sleeve is for single use only and must not be reused.

Application

After preparing the tooth according to the dental condition, use the sleeve as follows to prepare the PREDPOST+ posts.

Direction of the micro-applicator insertion into the sheath.

1. Insert the posts PREDPOST+ in the sleeve.
2. Insert the micro-applicator of a dual-cure luting composite onto the mixing tip. Expel some composite to remove any air bubbles.
3. Insert micro-applicator into the opening of the multi-post sleeve.
4. Pull on the sheath to uncover 1/3 of the length of the multi-post
5. Maintain the multi-post in position and inject the composite until it completely passes through the sleeve and exits.
6. Push the multi-post back into its sleeve while maintaining

pressure on the syringe.

7. Release pressure on multi-post while continuing to inject the composite, to ensure the multi-post is completely filled with composite.
8. Assemble and insert the PREDPOST+ according to PREDPOST+ instructions.

Cleaning and disinfection

- The sleeve is clean but not delivered sterile.

Guarantee – Limited liability

The guarantee is limited to the quality of the product: any defective product will be replaced. The Batch Number must be referenced in all correspondence asking to identify the product. This product has been developed for use in dentistry and must always be used according to the Directions. Any harm resulting from a failure to comply with these provisions, incorrect handling or use for purposes other than those indicated in the instructions will not be the manufacturer's responsibility. Before every use, the user is obliged to check the compatibility of the equipment with the intended application: in consequence, he or she is entirely liable for use of the product and any associated damages.

Warning

- Product use reserved for dental applications.
- Keep out of the reach of children.

Dispose of waste according to current regulations.

In the event of serious incidents, inform the competent authorities of the country.

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