[illegible]

**Norelfi Aragó**

## Monofilament in polyvinylidenefluoride Steriel en niet-absorbbaar chirurgisch hechtmiddel

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User Instructions

Norefil Aragó

Polyvinylidene fluoride monofilament  
Surgical, sterile and non-absorbable sutures

**Description:**  
Norefil Aragó is a sterile, monofilament and non-absorbable surgical suture composed of polyvinylidene fluoride monofilament. For better visibility, the thread has been dyed blue with Solvent Blue 104 (C.I. 61568) dye. It complies with the requirements of Sterile Non-Absorbable Fibres according to the European and American USP Pharmacopoeias.

**Indications:**  
Norefil Aragó is indicated for applications where, in accordance with surgical practice, non-absorbable materials are used for sutures. It is suitable for the approximation of soft tissue and ligatures, including use in cardiovascular tissue, plastic surgery, intestinal surgery and ophthalmology.

**Effects:**  
Norefil Aragó causes a minimal acute reaction of the tissue on which a gradual encapsulation occurs due to the connective tissue. It is resistant to traction, slides easily through tissue, has high chemical and biological inertness, and is knot-safe. It is non-absorbable and remains in the organism indefinitely.

**Contraindications:**  
Unknown.

**Warnings:**  
Norefil Aragó users must be familiar with surgical techniques and the use of non-absorbable sutures, because the risk of the injury reopening (dehiscence) depends on where it is used and the suture material employed. Do not re-sterilise/re-use. Do not use if the packaging is open. Re-using a single-use product creates a potential risk of infection for the patient. Product contamination may cause injury or disease, or lead to the death of the patient. Re-processing could jeopardise the structural integrity of the product or cause it to fail. Using the product without the label and instructions will result in using the product beyond its expiry date and the loss of its traceability. Altering the original packaging may jeopardise the sterility and functionality of the product.

**Precautions:**  
All sutures must be handled with care. Avoid damaging the suture by crushing or folding it through the use of forceps or needle holders. If needles are remodelled manually, they may lose their strength and become less resistant to bending and breaking. The surgeon must take into consideration the type of knot to be used in a surgical technique. When handling sutures, due precautions must be taken to prevent surgical instruments, such as forceps and needle holders, from damaging the suture material by crushing, folding or scraping it and/or deforming the needle. The user must be familiar with suture techniques and must handle the needles with

care so as to not be pricked by them. Discard used needles into a special container. Hold the needle between one-third (1/3) and a half (1/2) of the distance from the distal extremity to the tip of the needle. If it is held close to the tip, the capacity for penetration will decrease and this could also cause a fracture to the needle. If the needle is held by the distal extremity, the needle may bend or fracture.

**Side effects:**  
Mild reactions in the tissue surrounding the suture material may appear at the beginning. Like any foreign object, Norefil Aragó may exacerbate a prior infection. As with any foreign object, prolonged contact of this or any other suture with saline solutions such as those in urinary and biliary conduits may lead to the formation of calculus.

**Presentations:**  
This suture can be supplied in USP sizes 10/0 (EP 0.2) up to USP 2 (EP 5) in pre-cut threads of different lengths and with a variable number of threads, without a needle or swaged to different types of traumatic needles. See the catalogue to obtain more detailed information.

**Storage conditions:**  
Store at room temperature. prevent prolonged exposure to high temperatures and protect from dampness. Do not use after the expiry date.

Explanation of symbols on the packaging:

|  |                                                          |
|--|----------------------------------------------------------|
|  | Batch No                                                 |
|  | Catalogue reference                                      |
|  | Single use only                                          |
|  | Do not re-sterilise                                      |
|  | Expiry yyyy-mm-year/month                                |
|  | See instructions                                         |
|  | Sterilised by Gamma Rays                                 |
|  | Keep out of sunlight                                     |
|  | Do not use if the unit packaging is open or deteriorated |
|  | Manufacturer                                             |

CE Marking = identification number of the notified body. This product meets the essential requirements of the Council Directive 93/42/EEC concerning medical devices.

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