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P.G.A. Rapid Aragón

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User Instructions

P.G.A. Rapid Aragón

Braided and Coated fast-absorption Polyglycolic Acid Surgical Suture, sterile and absorbable

Description:
P.G.A. Rapid Aragón is a sterile, synthetic, braided and absorbable surgical suture based on Polyglycolic Acid (P.G.A.). The empirical formula is $(C_4H_8O_2)_n$. In order to enhance the sliding and handling properties, the braided thread has been covered by a mixture composed of stearic acid with a glycolic and lactac acid copolymer (30/70) or sucrose fatty acid esters and polycaprolactone. It is supplied colourless or dyed with DMC Violet Nº 2 dye (C.I.60725). It complies with the requirements of the European Pharmacopoeia for Sterile Absorbable Braided Synthetic Sutures and the American Pharmacopoeia (USP) with respect to Absorbable Synthetic Sutures.

Indications:
P.G.A. Rapid Aragón is indicated for applications where, in accordance with surgical practice, absorbable materials are used for sutures. It is suitable for the approximation of soft tissue and ligatures, where an injury requires short-term support and fast absorption of the suture. Thanks to its absorption profile, P.G.A. Rapid Aragón is indicated for epidermal sutures, paediatric surgery, episiotomy, phimosis, and closure of the oral mucosa, as well as conjunctival sutures (ophthalmic surgery), but not for use in cardiovascular or neurological tissues.

Effect:
P.G.A. Rapid Aragón causes a minimal acute reaction of the tissue on which a gradual encapsulation occurs due to the connective tissue. The gradual loss of resistance to breakage and eventual absorption of the P.G.A. Rapid Aragón is carried out through hydrolysis. The absorption begins with a loss of resistance to tension, followed by a loss of mass. Studies on implants in rats show the following profile:

Seven days after implanting the profile: 50% of its initial tension, and after 14 days this is practically zero. The total absorption concludes at approximately between 42-60 days.

Contraindications:
Since the P.G.A. Rapid Aragón is absorbable, it must not be used on injuries that depend on prolonged reinforcement with sutures.

Warnings:
P.G.A. Rapid Aragón users must be familiar with the surgical techniques and the use of absorbable sutures, because the risk of injury resulting (dehiscence) depends on where it is used and the suture material employed. Respectingly, the P.G.A. Rapid Aragón is not suitable for elderly or weak patients, or for those suffering from a slow healing of wounds. Since this is an absorbable suture, the surgeon must consider using a supplementary non-absorbable suture in the suturing of parts that may be subject to expansion, tension possibly and/or require additional support. Do not re-sterilize-re-use. Do not use if the packaging is open. Re-using a single-use product creates a potential risk of infection for the patient. Product contamination may cause injury or disease, and lead to the death of the patient. Re-processing could compromise the structural integrity of the product or could cause it to fail. Using the product without the label and instructions will result in using the product beyond its expiry date and the loss of its traceability. Altering the original packaging may jeopardize the sterility and functionality of the product.

Precautions:
Sutures that must remain on the skin for more than 7 days may cause local irritations, for which they must be cut or removed. In tissues with deficient blood supply, there could be a risk of the thread breaking or being slow absorption. All sutures must be handled with care. Avoid damaging the suture by crushing or folding it through the use of forceps or needle holders. If the needles are remodelled manually, they can lose their strength and become less

resistant to bending and breaking. The surgeon must take into consideration the type of knot to be used in a surgical technique. When handling sutures, due precautions must be taken to prevent surgical instruments such as forceps and needle holders from damaging the suture material by crushing, folding or scraping it, and/or deforming the needle. The user must be familiar with suture techniques and must handle the needles with care so as to not be pricked by them. Discard used needles into a special container. Hold the needle between one-third (1/3) and a half (1/2) of the distance from the distal extremity to the tip of the needle. If it is held closer to the tip, the capacity for penetration will decrease and this could also cause a fracture of the needle. If the needle is held by the distal extremity, the needle may bend or fracture.

Side effects:
Mild reactions in the tissue surrounding the suture material may appear at the beginning. As with any foreign body, the P.G.A. Rapid Aragón may exacerbate a prior infection. As with any foreign body, prolonged contact of this or any other suture with saline solutions such as those found in urinary and biliary conduits may lead to the formation of calculi.

Presentations:
This suture can be supplied in USP sizes 10/0 (EP 0.2) up to USP 6 (EP 8) in pre-cut threads of different lengths and with a variable number of threads, without a needle or sutured to different types of traumatic needles. See the catalogue for more detailed information.

Storage conditions:
Store at room temperature, avoid prolonged exposure to high temperatures and protect from dampness. Do not use after the expiry date.

Explanation of symbols on the packaging:

LOT	Batch No
REF	Catalogue reference
☒	Single use only
🚫	Do not re-sterilise
🕒	Expiry yyyy-mm -year/month
📖	See instructions
STERILIZ	Sterilized by Gamma Rays
☀	Keep out of sunlight
🚫	Do not use if the unit packaging is open or deteriorated
🏭	Manufacturer

CE Marking - identification number of the notified body. This product meets the essential requirements of the Council Directive 93/42/EEC concerning medical devices.

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