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Supolene Aragón

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User Instructions

Supolene Aragón

Braided Polyester Impregnated with Teflon
Surgical, sterile and non-absorbable suture

Description:
Supolene Aragón is a sterile, braided and non-absorbable surgical suture composed of a thread made multi-filament with anti-capillary treatment. For better visibility, the thread has been dyed green with D&C Green No.6 (C.I. 61565) dye. The polyester fibres have been impregnated with Teflon. It complies with the requirements of Sterile Non-Absorbable Fibres according to the European and American (USP) Pharmacopoeias.

Indications:
Supolene Aragón is indicated for applications where, in accordance with surgical practice, non-absorbable materials are used for sutures. It is suitable for the approximation of soft tissue and ligatures, including use in cardiovascular, ophthalmic, orthopaedic and neurosurgery.

Effect:
Supolene Aragón causes a minimal acute reaction of the tissue on which a gradual encapsulation occurs due to the connective tissue. It is resistant to traction, slides easily through tissue, has a high chemical and biological inertness, and is knot safe. It is non-absorbable and remains in the body indefinitely.

Contraindications:
None known.

Warnings:
Supolene Aragón users must be familiar with the surgical techniques and the use of non-absorbable sutures, because the risk of the injury reopening (dehiscence) depends on where sutures is used and the suture material employed. Do not re-sterilize/re-use. Do not use if the packaging is open. Re-using a single-use product creates a potential risk of infection for the patient. Product contamination may cause injury or disease, or lead to the death of the patient. Re-processing could jeopardise the structural integrity of the product or cause it to fail. Using the product without the label and instructions will result in using the product beyond its expiry date and the loss of its traceability. Altering the original packaging may jeopardise the sterility and functionality of the product.

Precautions:
All sutures must be handled with care. Avoid damaging the suture by crushing or folding it through the use of forceps or needle holders. If the needles are remodelled manually, they may lose their strength and become less resistant to bending and breaking. The surgeon must take into consideration the type of knot to be used in a surgical technique. When handling sutures, due precautions must be taken to prevent surgical instruments such as forceps and needle holders from damaging the suture material by crushing, folding or scraping it, and/or deforming the needle. The user must be familiar with suture techniques and must handle the needles with

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care so as to not be pricked by them. Discard used needles into a special container. Hold the needle between one-third (1/3) and a half (1/2) of the distance from the distal extremity to the tip of the needle. If it is held close to the tip, the capacity for penetration will decrease and this could also cause a fracture to the needle. If the needle is held by the distal extremity, the needle may bend or fracture.

Side effects:
Mild reactions in the tissue surrounding the suture material may appear at the beginning. Like any foreign body, Supolene Aragón may exacerbate a prior infection. As with any foreign body, prolonged contact of this or any other suture with saline solutions such as those found in urinary and biliary conduits may lead to the formation of calculi.

Presentations:
This suture can be supplied in USP sizes 10/0 (EP 0.2) up to USP 2 (EP 5) in pre-cut threads of different lengths and with a variable number of threads, without a needle or swaged to different types of atraumatic needles. See the catalogue for more detailed information.

Storage conditions:
Store at room temperature, avoid prolonged exposure to high temperatures and protect from dampness. Do not use after the expiry date.

Explanation of Symbols on the packaging:

LOT	Batch No
REF	Catalogue reference
	Single use only
	Do not re-sterilise
	Expiry yyyy-mm-year/month
	See instructions
	Sterilised by Gamma Rays
	Keep out of sunlight
	Do not use if the unit packaging is open or deteriorated
	Manufacturer

CE Marking + identification number of the notified body. This product meets the essential requirements of the Council Directive 93/42/EEC concerning medical devices.

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