

Instructions for Use

GB

Cavex ProphyPaste

Cavex ProphyPaste – Fine

Cavex ProphyPaste – Regular

MD



Product description

Cavex ProphyPaste is a professional prophylaxis paste used to clean the tooth surface, facilitate the removal of all exogenous deposits, and to polish the hard tissue including restorations and is available in two different RDA variations.

Cavex ProphyPaste Regular has a higher abrasiveness with an RDA of 130 to 150. Therefore it is perfect for thorough cleaning, while still maintaining a smooth finish. It comes in a light blue/green colour with mint flavour.

Cavex ProphyPaste Fine has a lower abrasiveness with an RDA of 50 to 60. Therefore it is perfect for the gentle cleaning with a high polishing effect and very well suitable for use on implants, crowns and bridges. It comes in a pink colour with a pleasant cranberry flavour.

Both pastes are pH neutral and contain fluoride, anti-staining and anti-bacterial components. The pastes come in a flip cap tube making the tubes easy to open and close with one hand. Due to the ideal consistency, the paste is easy to apply and doesn't splatter. After the treatment the paste can easily be rinsed away without leaving any residue, also making it an excellent pre-conditioner for whitening treatment.

Composition

The overall chemical composition of Cavex ProphyPaste as follows:

sorbitol, hydrated silica, water, glycerine, silica, erythritol, lithium magnesium sodium silicate, laponite, sodium hexametaphosphate, ethoxyethanol, flavour, cellulose gum, sodium saccharine, colour, titanium dioxide, xanthan gum, sodium fluoride

Contains: 0,22 % sodium fluoride (1000 ppm fluoride)

Packaging

Tubes 60 ml – 100 gram

Indication for use

Cavex ProphyPaste Fine & Regular is indicated for use as a tooth polish and cleaning treatment for the enamel by a dental professional.

Contra-indications & Warnings

- Not for use on patients with a known allergy to any of the components
- For fluoride the following adverse reactions are possible in individuals hypersensitive to fluoride: eczema, atopic dermatitis, urticaria, gastric distress, headache and weakness
- Instructions and regulations for storage and disposal of dental material waste must be observed.

Handling

- Fill cup/brush with paste and run at low speed over each tooth surface, making sure that the edge of the cup/brush goes below the free margin of the gingival sulcus.
- Repeat this procedure for each tooth until all exposed surfaces have been treated.
- Un-waxed Dental Floss may be used to remove any excess paste through interproximal areas.

Instructions for Use

Labelling

This information is in compliance with European legislation.

Intended Patient(s)

Cavex ProphyPaste is intended for any patient who needs a prophylaxes treatment as indicated by the medical specialist or hygienist.

Although there are no safety issues expected with the use Cavex ProphyPaste, based upon the fluoride concentration we do not advice the use for pregnant women and children < 6 years.

Intended User(s)

Cavex ProphyPaste will only be used by dental professionals (dentist, hygienists).

Clinical benefits

Cavex ProphyPaste contains no hard particles, making it completely safe for enamel and preventing the sensation of sand particles in the mouth by the patient after treatment. So, next to excellent cleaning and polishing results Cavex ProphyPaste is safe, easy to use and pleasant for the patient.

Claims

- The creamy consistency containing a blend of abrasive components, maximises the removal of plaque & stains without abrasion of the hard tissue.
- pH neutral, contains fluoride & anti-bacterial components
- Ideal consistency, no splattering
- Perfect pre-conditioning for whitening

Product characteristics and specifications

Characteristic	Target specifications		Release specifications		Unit
	Fine	Regular	Fine	Regular	
Appearance	Pink paste with cranberry flavour	Green paste with mint flavour	Pink paste with cranberry flavour	Green paste with mint flavour	
PH	7 – 8	7 – 8	6.5 – 8.5	6.5 – 8.5	
Viscosity	40 - 100	40 - 100	30 - 110	30 - 110	Pa.s
* Relative Dentin Abrasion (RDA)	Max 250	Max 250	50 – 60	130 -150	
* Relative Enamel Abrasion (REA)	Max 40	Max 40	< 5	< 5	
* Enamel Polishing Potential (EPP)			> 35	> 35	
* Pellicle Cleaning Ration (PCR)	Max 100	Max 100	> 85	> 85	

Instructions for Use

Accessories used in combination:

There is no restriction known to any standard available disposable prophylaxis angle/cup.

Special advice

According to EU Medical Device Regulation users/ patients are obliged to report serious incidents with a medical device to the manufacturer and to the competent authority of the country, the incident occurred.

Storage advice

Do not store at temperatures higher than 25°C (77°F) or lower than 0 °C (32°F). Do not use this material beyond its expiry date. Avoid direct exposure to sunlight. Close containers firmly after use and keep out of reach of children.

Disposal information

Dispose of in accordance with local regulations. Do not dispose of contents or partially emptied packaging in household waste or allow them to enter the sewage system.

Cavex Holland BV
Fustweg 5
2031CJ Haarlem
The Netherlands
T +31 23 5307700
dental@cavex.nl

Made in The Netherlands

Our technical advice, whether verbal, in writing or by way of trials, is given in good faith but without warranty, and this also applies where proprietary rights of third parties are involved. It does not release you from the obligation to test the products supplied by us as to their suitability for the intended processes and uses. The application, use and processing of the products are beyond our control and, therefore, entirely your own responsibility. Should, in spite of this liability be established for any damage, it will be limited to the value of the goods delivered by us and used by you. We will, of course, provide consistent quality of our products within the scope of our General Conditions of Sale and Delivery.

Version: BW910IFU.02

Date: 25-aug-2025