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CALCIUM HYDROXIDE PREPARATION

8 NEEDLES

Contents:
4 x 1.2 ml syringes CONTROL
8 pre-bent needles

FOR PROFESSIONAL USE BY DENTISTS

Revision: 09/2015

 La Maison Dentaire SA
Alte Churerstrasse 36
FL-9496 Balzers - LIECHTENSTEIN

• Bactericidal • Bacteriostatic • pH > 12 • Radiopaque

CONTROL is a smooth, creamy, non-setting calcium hydroxide preparation in an aqueous hydroxyethylcellulose base with barium sulfate. It is recommended for all clinical applications where calcium hydroxide paste is indicated. It is a dressing, not a hard base.

Indications: Direct Pulp Capping, Pulpal Curettage, Pulpotomy, Dentin Bridge Formation, Temporary and Intermediate Canal Treatment, Periapical Cysts, Abscesses and Lesions, Root Resorption and Apexification

Note: Use a new applicator tip for each patient. To extend shelf life of CONTROL place a drop of water in the cap and recap syringe promptly after each use.

Note: Multi-dose syringes should either be encased in a fresh protective barrier for each patient or cleaned and disinfected between patients, as appropriate.

VITAL PULP TREATMENT

CONTROL is applied directly on the pulp in all pulp capping, pulpotomy and pulpal curettage procedures. It stimulates the pulp to form a new dentin bridge which is visible clinically after 2 - 3 months.

Directions for Direct Pulp Capping:

- Place a 22-gauge applicator tip on the CONTROL syringe and apply a very thin film of CONTROL on all exposed dentin and dry with a slow flow of air.
- Place a bead of CONTROL 3mm thick directly over the exposed pulp and dry with a slow flow of air until a crust appears. CONTROL does not set hard.
- Gently tamp the paste with a pellet of cotton to make sure it is in contact with the pulp.
- Place a thin mix of hard setting cement over CONTROL.
- After cement hardens, place final restoration.

Note: For vital pulpotomy and pulpal curettage procedures, pack pulp gently with sterile cotton until blood clots, then proceed as above. A local anesthetic may be used.

ROOT CANAL TREATMENT

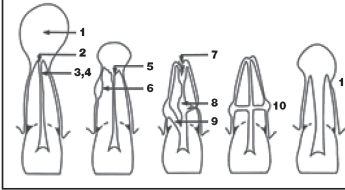
Use routinely as a bactericidal, bacteriostatic canal dressing between off visits and as a dressing for cysts, abscesses, lesions, etc. Also use for apexification and other hard tissue formation.

Directions for Root Canal Therapy:

- Prepare and cleanse canal in conventional manner.
- Place a 22-gauge applicator tip on the CONTROL syringe and fill canal with CONTROL.
- Seal access opening with your preferred cement.
- CONTROL is easily removed with file and irrigation.
- In cases requiring extended root canal therapy, change dressing every month.

Precautions: May be irritating to eyes, skin, respiratory system, gastrointestinal tract or mucous membranes, especially in large quantities and/or with prolonged exposure. For use by dental professionals only. Wear gloves, safety glasses; wash hands after use. Avoid unnecessary exposure.

CLINICAL SITUATIONS WHERE
CALCIUM HYDROXIDE CAN BE USED

- exudation control
- large periapical lesions
- antibacterial dressing
- temporary root filling
- apical inflammatory resorption
- inflammatory resorption following trauma
- apical internal resorption
- internal-external root resorption
- perforations
- transverse root fractures
- incompletely developed pulpless teeth
(Heithersay, G.S. Calcium Hydroxide in the
Treatment of Pulpless Teeth with Associated
Pathology, J. Brit Endo Society, 8[2]: 74-93 July 1975)

Packaging: Syringes (4 x 1.2ml + 8 applicator tips)

 shelf life is indicated on outer package

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