Rocanal®

Permanent Vital R2

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TAL is not reabsorbed in the canal system. In case of overfilling, it is slowly reabsorbed only in the periapical region by macrophages present in the living tissue which gradually eliminate this excess.

OPERATIVE PROCEDURES

- Diagnosis:** Vitality control (pulp-tester; criotest etc.). X-ray.
- Anaesthesia.** (anesthesia or devitalisation at the previous session)
- Isolate the tooth with a coffer dam.**
- Cleaning of the pulp chamber and location of the canal(s).**
- Removal of pulp and debridement of canal(s):**
 - Mix powder and liquid of ROCANAL PERMANENT VITAL on the glass slab.
 - Remove the pulp using the proper sterile instrument. Thoroughly irrigate with ROCANAL Irrigation, rinse and dry.
 - Reach the apex (dentine-cement limit or physiological foramen) and take a control X-ray with the canal instrument in place. Thus the exact measurement of the preparation is ensured.
 - Debride the canal with sterilized , adequate instruments. Thoroughly irrigate with ROCANAL Irrigation, rinse and dry with paper points.
- Root canal filling with ROCANAL PERMANENT VITAL.**

In the same session (or in a subsequent one) the canal filling is performed with ROCANAL PERMANENT VITAL. Pulpectomy on vital tooth being performed, the filling should be just short of the apex (as far as dentine-cement limit or physiological foramen). In the case of accidental overfilling with ROCANAL PERMANENT VITAL through the apex, leave the paste there without any other intervention. It will be absorbed by the patient without any adverse reaction. There is no interference with healing. For the filling of the canal, use power- driven "paste-filler" called also "lentulo-filler". With a very small quantity of ROCANAL PERMANENT VITAL on the spirals of the filler, insert the instrument (in "off" position) into the canal. Control if apex has been reached by referring also to the X-ray already taken. Start rotating in a clockwise direction and at moderate speed, withdrawing at the same time slowly the paste-filler. Repeat the operation with a new quantity of paste, for a second and

a third time, until the filling of the canal is completed. Remove any excess of the paste from pulp chamber. If filling is difficult, use a reamer charged with ROCANAL PERMANENT VITAL and rotate in opposite direction.

- X-ray control.**

If the filling is not complete, repeat above operation. For the filling some authors suggest the use of a power-driven root canal reamer, in a counter clockwise direction.

- Seal the tooth.**

MIXING INSTRUCTIONS FOR ROCANAL R2:

- Use sterile gloves, use sterile mixing slab and two single-end flexible sterile spatulas (one for powder, one formixing).
- Put one or two drops of liquid and small quantity of powder on the slab.
- Mix powder slowly with the liquid, with some pressure on the spatula.
- Mix as long as needed to obtain a creamy, crystal-free mixture (crystals present in the powder are soluble in the liquid).
- Proceed with your usual filling technique.
- Use sterile material
- The average time of hardening of the mixture is 2 to 4 minutes depending on the powder/liquid ratio and the temperature.

PRECAUTIONS FOR USE

Eliminate balances of unused product with the official rules in force. For dental professional use only.

SPECIAL STORAGE CONDITIONS

+8° +25°

keep in a cool place : +8° – +25°C

shelf life is indicated on the packaging

REF

0470

EC REP / Importer
La Maison Dentaire SA
Alte Churerstrasse 36
FL-9496 Balzers · Liechtenstein

Medirel SA
via Redondello 20, CH - 6982 Agno
Tel. +41 (0) 91 604 5434
Fax +41 (0) 91 605 3026
info@medirel.ch www.medirel.ch

1th certification CE 1998

REV.: 05/2021

Rocanal®


Permanent Vital R2

A permanent filling cement for the root canal system to be used after pulpectomy of a vital tooth


COMPOSITION

Alcoholic solution of eugenol:	Powder:
Eugenol, Rosin, Castor oil, Glycerol, Alcohol	Zinc oxide, Titanium oxide, Zinc stearate, Zinc acetate, Orthophenyphenol (OPP), Calcium wolframate, Anise

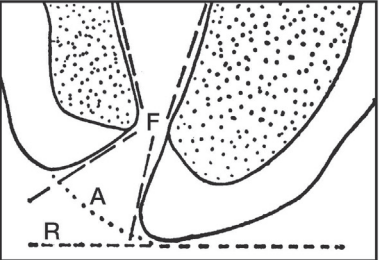to prepare just before use (see mixing instructions hereafter)

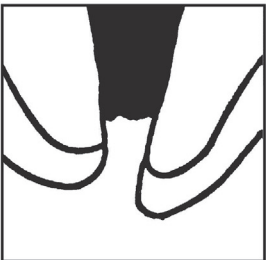F = physiological foramen
A = anatomical foramen
R = radiological foramen

Root canal filling after vital pulpectomy (just short of the apex)

INDICATIONS

A permanent filling cement for the root canal system to be used after pulpectomie of a vital tooth, in adolescents over 12 years of age (at least 30 kg) and adults.

It is advisable to treat one tooth per day with a maximum of 5 treatments per year for adolescents over 12 years and 10 treatments per year for adults.

CONTRA INDICATIONS/SIDE EFFECTS

Not indicated in case of intolerance or allergy to one of the components and in particular in case of allergy or suspected allergy to Eugenol. A temporary periapical parodontitis may appear after applying the product and resolves in two days with proper treatment. No side effects have been reported to date.

NOTE

After vital pulpectomy it is necessary to fill the root canal system with a proper root canal cement. The filling should be just short of the apex (as far as dentine-cement limit or physiological foramen). In vital teeth, the apical zone of the canal is vital and has a self defence against infection. It is proved that in this zone, the pulp is sterile. To irritate the apical or periapical zones by any mechanical or chemical means, is irrational. To irrigate this zone with strong and irritating antiseptics, will only delay process of healing and biological closing of the apex. ROCANAL PERMANENT VITAL has been developed for the root canal filling after vital pulpectomy. This cement most closely meets the requirements of an ideal filling. It realizes a physical adhesion to the dentine walls. ROCANAL PERMANENT VITAL is ideally diffused especially in its state of softness as it enters the canal. Equilibrium is established after hardening. The cement hardens completely in the canal ; it is stable and insoluble in tissue fluids. It is not supportive of bacterial growth. Pulp remnants which always remain after total pulpectomy, are protected against infection as well as putrescence. In case of overfilling into the periapical zone, it is non-irritating to periapical tissue. It is radiopaque and a non conductor of heat. Does not discolor the tooth. In case of doubtful diagnosis (vital or gangrenous tooth), you can leave Calcium Hydroxide (Control ®) in the tooth and fill it temporarily. Fill the tooth in a subsequent session with ROCANAL PERMANENT VITAL.

Attention: Partial pulp removal (mid-way or less) followed by a partial filling of the canal, will inevitably lead to a failure. ROCANAL PERMANENT VITAL results from the most recent researches in Endodontics. The action is studied so as to induce no painful reaction. ROCANAL PERMANENT VI-

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