INSTRUCTIONS FOR USE

Cavex Boil&Bite

Description
Cavex Boil&Bite is a double-sided thermoplastic ("boil & bite") tray which serves as upper and lower individual tray at the same time.

Packaging
8 x Cavex Boil&Bite / box

Composition
Cavex Boil&Bite is manufactured from a food grade PE copolymer compound with a hardness of 86 Shore A.

Indications for use
Tooth whitening, conditioning, desensitizing or fluoride application.

Handling procedure
1. Bring some water to the boil.
2. Place the Cavex Boil&Bite in a suitable (heat-resistant) glass.
3. Add enough water to the Cavex Boil&Bite to cover the tray - with the exception of the handle.
4. Leave the unit submerged in the water for one minute.
5. Carefully remove the Cavex Boil&Bite from the hot water.
6. Shake off any surplus water.
7. Put the Cavex Boil&Bite into your mouth immediately and adjust the shape of the tray by pressing it firmly against the teeth with your fingers. Do this for both the top and bottom teeth.

Contra-Indications
None.

Warnings & first aid instructions
Hot (boiling) water can cause burns.

Storage advice
Store in a cool place, dry place and protected from direct sunlight.

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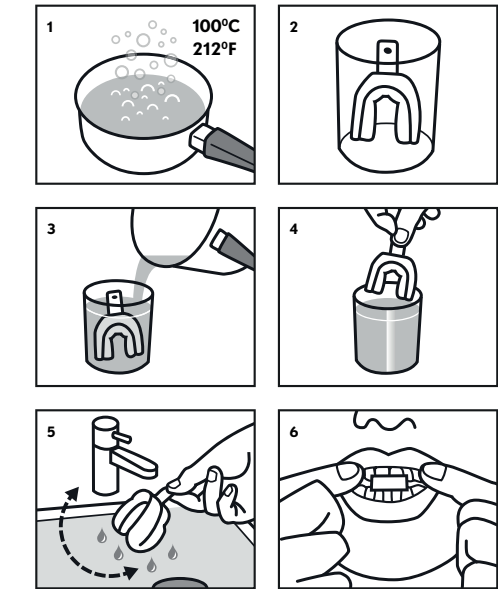
Our technical advice, whether verbal, in writing or by way of trials, is given in good faith but without warranty, and this also applies where proprietary rights of third parties are involved. It does not release you from the obligation to test the products supplied by us as to their suitability for the intended processes and uses.
The application, use and processing of the products are beyond our control and, therefore, entirely your own responsibility. Should, in spite of this liability be established for any damage, it will be limited to the value of the goods delivered by us and used by you. We will, of course, provide consistent quality of our products within the scope of our General Conditions of Sale and Delivery.

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CAVEYOUR IMPRESSION IS OUR CONCERN

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